

VETERINARY PRODUCTS LABORATORIES

INNOVATIVE PRODUCTS IN VETERINARY MEDICINE™

CALL TOLL FREE 1.888.241.9545



TISSUMEND II and TISSUMEND II STERILE Storage and Handling Instructions

TISSUMEND II and TISSUMEND II STERILE Tissue Adhesives need to be stored under refrigerated conditions.

(a) **GENERAL STORAGE:**

TISSUMEND II and TISSUMEND II STERILE are very stable products, but proper storage instructions will greatly improve the products' performance. Therefore, it is recommended that TISSUMEND II and TISSUMEND II STERILE are stored under refrigerated conditions.

(b) **SHIPPING UNDER HOT CONDITIONS:**

In transit during the summer months where temperatures exceed 90°F, we recommend that TISSUMEND II and TISSUMEND II STERILE be shipped under refrigerated conditions to ensure the veterinarian receives the most stable product.

(c) **GENERAL SHIPPING:**

We recommend that if transit time exceeds 4 days duration then the product should be shipped under refrigeration as well.

As with most liquids, their stability compared to solid materials is often more variable and care must be taken to avoid extreme temperature fluctuations. These temperature fluctuations may result in condensation within the pipette and alter performance of the product. Therefore, rapid heating-cooling cycles are not recommended.